Instructions manual

Ergo

1028

Podiatry Stool

VERSION 1.24

Weelkome!

Thank you for having acquired a WEELKO product and for the trust shown in us.

This beauty instrument has been designed to fulfill the most demanding work requirements of those professionals working in the beauty and aesthetics field, for face beauty treatments. This instrument should only be used by trained and skilled beauticians. This device should never be used for any purpose than its intended design limitations.

The company reserves the right to modify this manual with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail to info@weelko.com informing us so that we can correct them.

Use of the manual

Before using this beauty instrument, please read the instruction manual carefully. It is important to keep this manual and store it in a safe place so that it can be consulted at any time.

If you follow these instructions carefully and consider the safety measures detailed, you will avoid any problems when using the device.

Some texts and/or illustrations may not belong to your instrument, since they are general, and they may depend on your product choice at the purchase time.

Our company reserves the right to improve the design of the product and modify them without prior

Warranty

The warranty period is effective from the date of purchase of the product. It covers all the components of the beauty instrument if the product is used under normal conditions and correct use. Please contact your point of sale to get further information on warranty terms and conditions. The warranty does not cover defects or malfunction caused by:

- Misuse, abuse or improper maintenance of the instrument.
- Non-observance of the precautions and warnings stated in this instruction manual.
- Lack of maintenance and maintenance or repair tasks carried out by non-qualified service personnel who are unauthorized by the manufacturer.
- Problems or breakdowns that may appear after non-authorized repairs performed by third parties, especially non-authorized personnel.
- Wrong and faulty electrical installations or which do not fulfil the legal requirements.
- The company is not liable for incidental or consequential damage to property or business.

EC Declaration of conformity

This product is produced and marked in conformity with the directives hereby stated:

Low voltage Directive: 2014/35/CE
Electromagnetic compatibility : 2014/30/CE

Waste management

The symbol of the crossed-out container means that it is legally required to dispose of the product separately from unsorted urban waste. It means that it must be disposed of separately from household waste.

The proper disposal of these products contributes to saving valuable resources and prevents negative effects on human health and the environment if not discarded properly.

For their proper treatment, recovery, and recycling, take these products to designated collection points or consult the local laws of your country.

Safety instructions

. This product is intended for use by qualified and certified professionals only.

. Tighten bolts and nuts regularly to prevent them from loosening, slipping, vibrating, or falling off.

. Do not use the product if it is damaged or has any faults. If you find any problems or abnormal sounds, inspect the stool immediately and use it again only after the problem has been resolved.

. The user must be familiar with the precautions described in each section of the manual to ensure the safety of people and the product.

. If the wheels are damaged, please replace them before use.

. After each treatment, use a neutral solution to clean any dust or stains present on the product.

. Place the product in a suitable location, according to the overall layout, natural or artificial lighting, and working environment conditions.

. Keep this product out of the reach of children and do not allow them to use it.


. Do not modify, disassemble, or repair this product yourself. When a repair is necessary, contact the manufacturer, agent, or professional maintenance personnel to inspect and repair it.

Installation

No.	1028	Quantity
1	Plastic 5-star base with wheels	1
2	Gas lift cover	1
3	Ø 167 Base plate (short handle)	1
4	Armrest (fixed part)	1
5	Seat (cushion + rotary disk)	1
6	Gaslift	1
7	B002 Spanner	1
8	Allen wrench (5mm)	1
9	Hexagon socket head screws (M6 x16)	4
10	Hexagon socket head screws (M6x20)	4
11	M6 Hexagon self-locking nuts	4
12	Ø16 flat washer	8
13	Ø16 spring washer	8

Assembly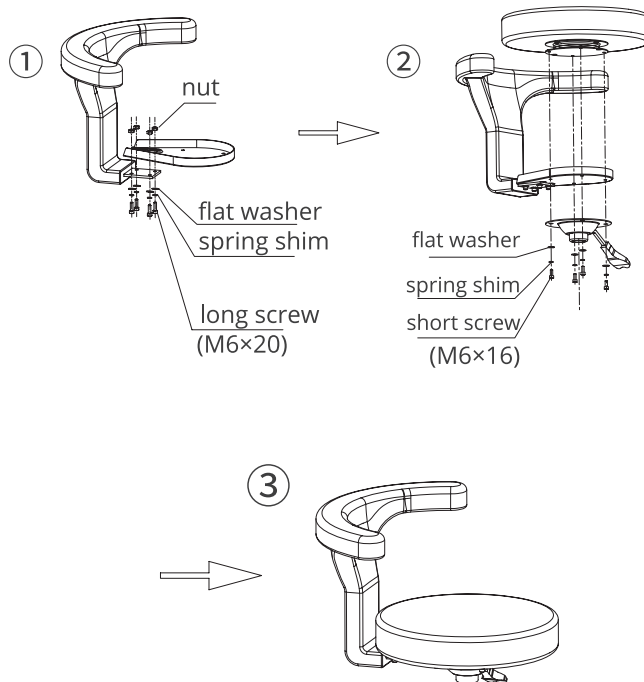

Assembly



Product description

Innovative pedicure and podiatry stool with an anatomical U-shaped revolving backrest up to 360° to support both back and elbows.


④ 1028

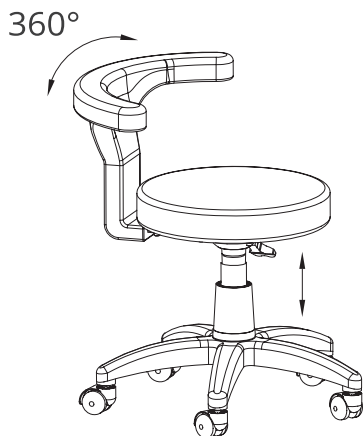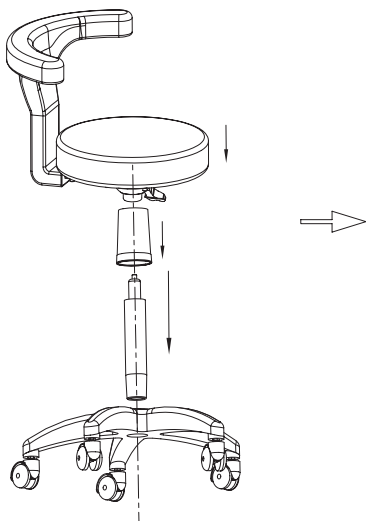Dimensions

1028

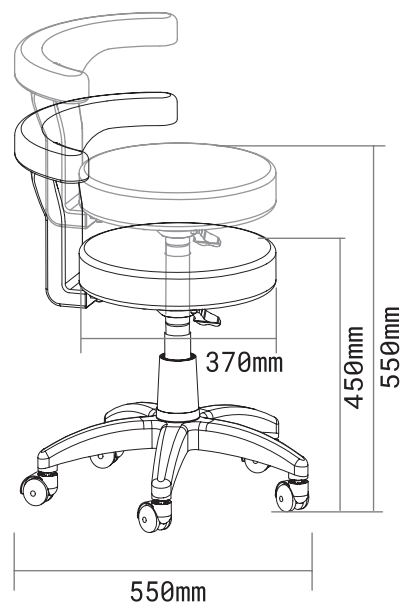Technical sheet

Reference of the product ERGO	1028
Packaging dimensions (mm)	590 x 300 x 600 mm
Lifting capacity	150 Kg
Net weight	9.3 kg
Gross weight	11.2 kg

Technical service

Only a technician authorised by Weelko is allowed to perform any kind of repairs on the beauty device. Please contact your distributor or point of sales for further information on technical assistance.

Troubleshooting

Fault	Reason	Solution
The stool is not working properly.	1. It doesn't go up or down.	1. Replace the gas lift.
	2. Casters don't turn or move.	2. Replace the casters.

Pol. Ind. La Vallan
Camí de Garreta, 92
08600 Berga
Barcelona - Spain
T. +34 93 824 90 20
E. info@weelko.com

Our products are subject
to continuous technical
improvements. Weelko reserves
the right to change or modify this
manual and its contents without
prior notice.

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