Manual de instrucciones

B-Equipment

F312B

Aparato de ultrasonidos.

VERSION 1.24

Weelkome!

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VERSION 1.24

Weelkome!

Thank you for having acquired a WEELKO product and for the trust shown in us.

This beauty instrument has been designed to fulfill the most demanding work requirements of those professionals working in the beauty and aesthetics field, for face beauty treatments. This instrument should only be used by trained and skilled beauticians. This device should never be used for any purpose than its intended design limitations.

The company reserves the right to modify this manual with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail to info@weelko.com informing us so that we can correct them.

Use of the manual

Before using this beauty instrument, please read the instruction manual carefully. It is important to keep this manual and store it in a safe place so that it can be consulted at any time.

If you follow these instructions carefully and consider the safety measures detailed, you will avoid any problems when using the device.

Some texts and/or illustrations may not belong to your instrument, since they are general, and they may depend on your product choice at the purchase time.

Our company reserves the right to improve the design of the product and modify them without prior

Warranty

The warranty period is effective from the date of purchase of the product. It covers all the components of the beauty instrument if the product is used under normal conditions and correct use. Please contact your point of sale to get further information on warranty terms and conditions. The warranty does not cover defects or malfunction caused by:

- Misuse, abuse or improper maintenance of the instrument.
- Non-observance of the precautions and warnings stated in this instruction manual.
- Lack of maintenance and maintenance or repair tasks carried out by non-qualified service personnel who are unauthorized by the manufacturer.
- Problems or breakdowns that may appear after non-authorized repairs performed by third parties, especially non-authorized personnel.
- Wrong and faulty electrical installations or which do not fulfil the legal requirements.
- The company is not liable for incidental or consequential damage to property or business.

EC Declaration of conformity

This product is produced and marked in conformity with the directives hereby stated:

Low voltage Directive: 2014/35/CE
Electromagnetic compatibility : 2014/30/CE

Waste management

The symbol of the crossed-out container means that it is legally required to dispose of the product separately from unsorted urban waste. It means that it must be disposed of separately from household waste.

The proper disposal of these products contributes to saving valuable resources and prevents negative effects on human health and the environment if not discarded properly.

For their proper treatment, recovery, and recycling, take these products to designated collection points or consult the local laws of your country.

Safety instructions

- This product is intended only for qualified and certified beauty practitioners.
- Do not use or store the product in a damp place such as bathroom to avoid malfunction and danger.
- Do not use this product near flammable items (e.g., alcohol, nail polish remover, sprays, etc.)
- Do not use this product under the following conditions: when the power cord or plug is damaged or when it gets hot, and when the plug in the power socket comes loose.
- Do not plug and unplug power cord connectors or use this product with wet hands.
- Please make sure that the working voltage of this product matches its rated voltage.
- Before its use, make sure that the power plug and the power cord connector are fully plugged in to avoid any danger.
- If the device does not have to be used, please turn it off and disconnect it from the power supply.
- Please regularly clean the dust accumulated on the power plug and power cord to avoid failure of insulation caused by moisture, dust, and any other factors.
- Be sure to use the special power cord assembly for this product. Do not use the power cord assembly on any other appliance.
- To keep the power cord in best condition, do not wrap it around the product.
- Keep this product out of reach of children. Do not allow them to use this product.
- Please do not modify, disassemble, or repair this product by yourself. When a repair is needed, please contact the manufacturer, agent, or professional maintenance personnel to check and repair it.
- If the plug is damaged and it must be repaired or replaced, please contact the distributor, the service center or related personal with appropriate skills and experience.
- Please be sure to use this product according to the purpose specified in this manual. Do not use accessories which are not recommended by the manufacturer.

Safety during the treatment

- Do not point the probe directly at the eyeball, touch the eyes or massage the upper eyelid.
- The use of this device is completely prohibited for the following people:
 - Pregnant women and those equipped with a cardiac chronometer;
 - Tumor patients and serious heart disease;
 - Skin burns, scalds, sunburn, inflammation or other wounds;
 - Allergy patients and people with contact dermatitis.
- Before care, it is necessary to apply enough cream or essence on the care part to ensure that there is a good medium between the probe and the skin, so as not to pull the skin too much.
- When using, the probe should be close to the skin to prevent cavitation and damage to the skin.
- When used, the probe should not be in the output state without operation, in case of damage to the probe or reduce the service life of the probe.
- If the probe is too hot, it will burn the skin, so the use time should not exceed 15 minutes, sensitive skin should reduce the use time and energy intensity. Lengthening the use time will not enhance the nursing effect and may lead to danger. Attention should be paid to the care of a skin stay time should not exceed 5 seconds.
- After nursing, the skin will appear a short fever ruddy phenomenon, due to the vibration of the probe and heat production, increasing blood circulation, which belongs to the normal state.
- If there is any abnormal situation, please contact the dealer and ask a professional maintenance service to check the device. Do not open the machine frame by yourself and without authorization.


Product description


Ultrasound instrument with a facial and an orbicular probe. The package contains: main unit + 2 shelves + 1 stainless steel ultrasonic tip plane probe + 1 stainless steel ultrasonic plane probe + 1 power cord.

Screen & functions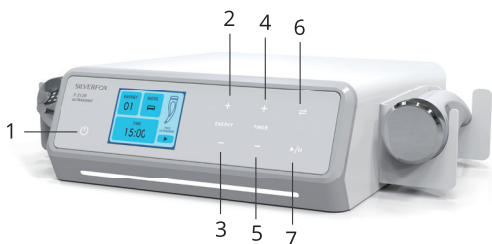

1. Power switch
2. Power Energy (Increase +)
3. Power Energy (Decrease -)
4. Time increase +
5. Time increase -
6. Mode switch
7. Start/Pause

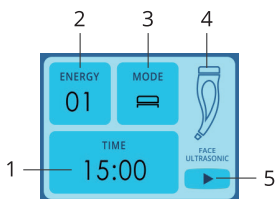1. Time
2. Energy level
3. Probe Mode Option
4. Working status
5. Operating status

Operation

1. Insert the ultrasound probe into the corresponding socket of the Beauty Equipment. When the equipment is connected to the power cable properly, the "Power switch" is displayed in red. Press the "Power Switch" button to enter into the working mode. The screen will be displayed in white colour. The equipment is set by default at EYE PROBE output state; thus the energy output of level 1 is set to 15 minutes.

2. Different modes can be selected according to your needs. Press "Mode switch" to choose the appropriate output mode (Positive, negative and mixed output mode)

3. Press the "Time increase/decrease button" which allows you to set the Working time from 1 to 30 minutes.

4. Press the "Start"/"Pause" button to start the operation. The power energy level is adjustable and allows to be set from level 1 to 10.

5. When you want the Ultrasound probe to stop working during the operation you can press the "Pause" button.

7. Press the "power switch" to turn off the equipment when the treatment ends, and the "power switch" will display red at this time. Clean the ultrasound probe with 75% alcohol after unplugging the power plug, and store in a dry place.

Recommended Using time

The treatment should not exceed 15 minutes with normal skin condition.

Warning: Do not put the metal part of the probe in contact with water.

Cosmetic purpose

The application of ultrasounds favors the reduction of ridges and scars, contributes to skin hydration and helps cosmetic actives to be well absorbed into the skin. The treatment with ultrasounds improves blood circulation and increases cell regeneration. The Weelko device allows choosing the type of wave: continuous for normal type skin and thicker tissue areas; combined wave for smoother skin areas and intermittent for sensitive skins.

Technical sheet

B-Equipment reference	F-312B
Packaging dimensions (cm)	37 x 27,5 x 14 cm
Function	Ultrasound device
Power	220-240 V
Frequency	50/60Hz
Rated power	11W

Assistance

Only a technician authorised by Weelko is allowed to perform any kind of repairs on the bed. Please contact your distributor or point of sales for further information on technical assistance.

Troubleshooting

This troubleshooting guide is general for all the instruments belonging to F31X new series. Some of these faults and solutions may not apply to your device.

Fault	Reason	Solution
The main machine does not work	1. No power	1. Check whether the fuse in the power base is burned out. If burnt, replace the fuse. 2. Press the start button, the button will light up after it is powered.
	2. The power cord is not properly inserted or connected	2. Plug or connect the power cord properly.
The display screen does not work	1. No power	1. Turn on the power button (ON). 2. Contact your supplier.
The accessory probe does not work	1. Poor interface contact	1. Unplug the output cable, and then reinsert the socket.
Weak power for Cold & Hot hammer, scrubber, RF, Galvanic and other functions	1. The connector is not properly plugged in. 2. The accessories are too hot. Overheating can cause weak output power.	1. Adjust power to a higher level. 2. Wait for the accessories to cool down. Turn off the device when not in use.
Dermabrasion has weak suction	1. The thread of the diamond tip is not properly tightened. 2. The inside of the dermabrasion handle is obstructed. 3. The air tube is worn out. 4. The connection between the air tube and the suction handle is loose.	1. Adjust the power and the air regulation valve. 2. Replace the microdermabrasion accessories. 3. Replace the air hose with a new one, adjust the connecting piece, and change the filters.

Troubleshooting

Fault	Reason	Solution
Sprayer with weak spraying effect	<ol style="list-style-type: none"> 1. The joining screw of the airbrush with the spray bottle is not properly tightened. 2. The water receptacle is blocked. 3. The air tube is worn out. 4. The connection between the air tube and the suction handle is loose. 5. There is not enough water in the tank. 6. The hose is bent. 7. The vacuum is not connected. 	<ol style="list-style-type: none"> 2. Adjust the spray level to a higher one. 3. Replace the water receptacle. 4. Replace the air hose with a new one and adjust the connecting piece. 5. Check the water level in the bottle and add more when necessary. 6. Connect both handles.
The suction of the pen is weak	<ol style="list-style-type: none"> 1. The glass nozzle is not properly placed. 2. The suction handle is obstructed. 3. The air hose is worn out, damaged, or bent. 4. The connecting piece between the air hose and the handle has loosened. 	<ol style="list-style-type: none"> 1. Adjust the spray level to a higher one. 2. Replace the suction handle. 3. Replace the air hose with a new one and adjust the connecting piece.
The output of high frequency is weak	<ol style="list-style-type: none"> 1. The electrode is not properly placed. 2. Wear of the electrode rod due to excessive use. 	<ol style="list-style-type: none"> 1. Adjust the power to a higher level. 2. Replace the high-frequency accessories.

Pol. Ind. La Vallan
Camí de Garreta, 92
08600 Berga
Barcelona - Spain
T. +34 93 824 90 20
E. info@weelko.com

Our products are subject
to continuous technical
improvements. Weelko reserves
the right to change or modify this
manual and its content without
prior notice.