EN

WKB008  
Instruction Manual  
WE-2022-WKB008-1


# ELECTROSTIMULATION

The company reserves the right to amend the details of the instruments with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

## PACKAGE CONTENTS


- These components are subject to change without prior notice.
- Appearance of product may differ from the image.

No.	PRODUCT	REF.	CANT.
1	BIG ELECTRODES	-	10
2	SMALL ELECTRODES	-	10
3	CABLES WITH 10 OUTPUT CONNECTORS	-	2
4	ELASTIC BELTS WITH VELCRO	-	20
5	MAIN UNIT	-	1
6	POWER CORD	-	1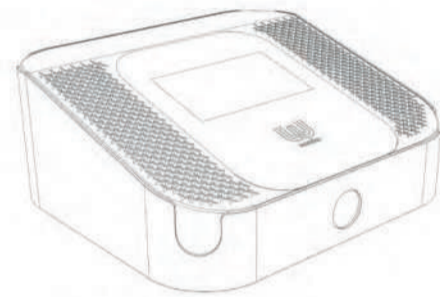

## INSTRUCTIONS OF USE

1. Connect the power cord via the rear panel.
2. Connect the cables of the electrodes properly to the rear panel. Follow the numerical instructions, connector with cables from 5 to 9 is to be plugged on the left and from 0 to 4 to be plugged on the right.
3. Clean the skin with some alcohol before placing the electrodes. Place them on the desired areas: small electrodes are for small areas such as arms, abdomen and shoulders and big ones for legs and back.
8. The "LEVEL" menu allows to adjust intensity. It can be independently adjusted on each electrode from 1 to 15. In case of programs C and D, the intensity is enough for a functional activity until the contraction is visible.
9. The "TIME" menu allows to adjust the treatment time. In case of programs A and B the recommended time is 45 min. In case of programs C and D, the recommended time is 20-30 min. maximum.
10. Press "PLAY/PAUSE" to begin or pause the treatment.

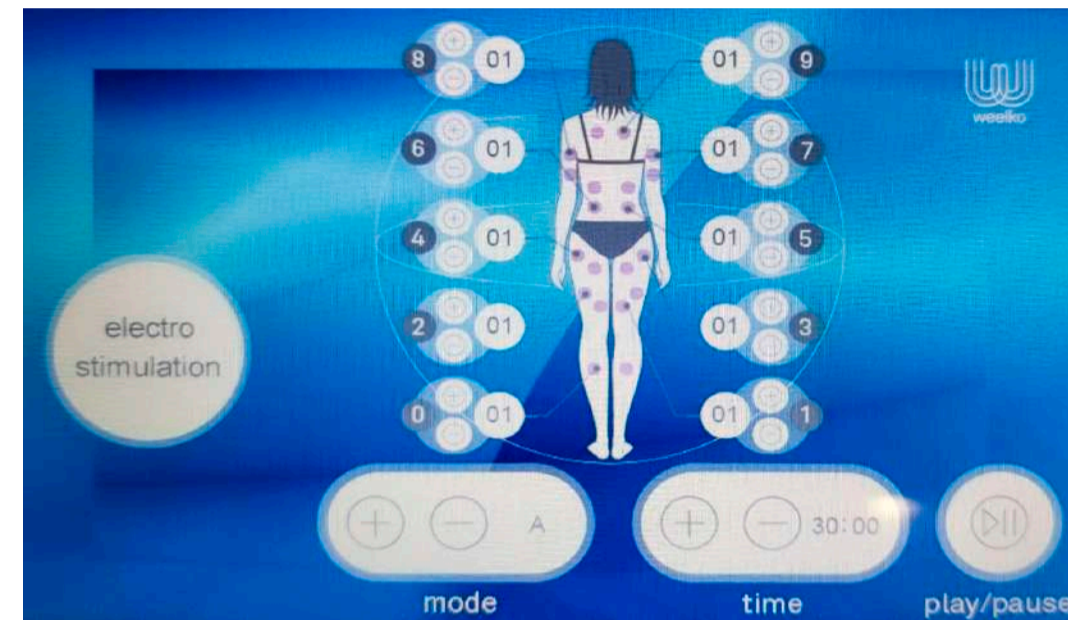## CAUTIONS

- During the treatment, check with the patient if the current is appropriate.
- Make sure the current is at the minimum intensity before pressing on PLAY and increase intensity very progressively.
- Do not remove the electrodes from the skin before turning the machine completely off.

## MAINTENANCE

- Connect and disconnect the electrodes carefully to extend the product lifetime. Only the general power cord at the rear part of the device should be connected and disconnected.
- After use, clean the electrodes with a wet cloth and some disinfecting product.

## ACCESSORIES

REF.	DESCRIPCIÓN
-	EMS BIG ELECTRODES
-	EMS SMALL ELECTRODES
-	EMS CABLE
-	EMS ELASTIC BELTS

## WARNING

- Never under any circumstances attempt to open or inspect the internal components or accessories of the machine. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.
- Never use the unit in humid or poorly ventilated environments, such as wet rooms, bathrooms, etc. to prevent damage to the electrical components of your unit.
- Never use the unit outdoors; exposure to the elements can cause damage.
- If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards, or contact the dealer for service.
- Do not use on pregnant women, on people with pace-makers (any electrical or rhythmic heart-regulating device) or with skin infections.

## EC-DECLARATION OF COMPLIANCE

This product meets the following requirements:




1. Conditions set out in the Low Voltage Directive: 2014/35/EU
2. Conditions set out in the Electromagnetic Compatibility Directive: 2014/30/EU

## RECYCLING

 Whenever any of our electrical products or their presentation packaging bear this symbol, it means that these products should not be treated as conventional household waste in Europe. To ensure proper treatment of this waste, dispose of it in accordance with local laws or as required for the disposal of electrical equipment. This initiative will help preserve save natural resources and improve standards of environmental protection in regard to the treatment of electrical waste.

## WELCOME!

Thank you very much for purchase. Please read the instructions carefully before operating the instrument. You can also ask a professional for guidance to make sure you are using the instrument correctly. We will accept no responsibility for any accidents that occur due to incorrect operation.

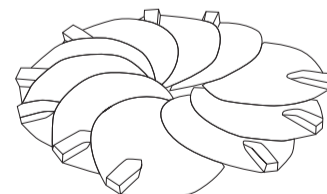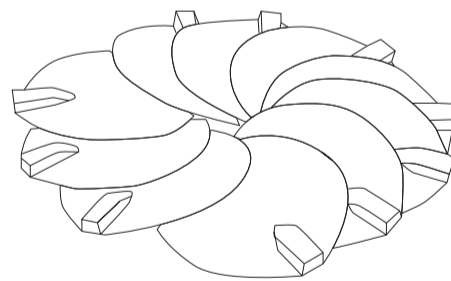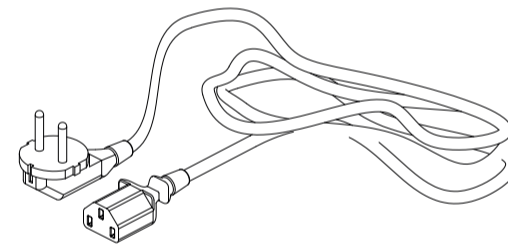4. The red cable is the positive pole and has to be placed on the motor point or the muscle belly. The black cable will be the negative pole and will be placed on the electrode close to the muscle insertion. Use conductive gel on the electrode before placing them in contact with the skin.

5. Use the elastic belts to hold the electrodes in touch with the skin.

6. Turn on the power button on the front panel and click on Weelko logo to select the desired menus.

7. Select "electrostimulation" menu to start with the treatment.

## TECHNICAL SHEET

Weights and sizes	Packing size (cm)	52x45x23
	Gross weight (Kg)	8
	Product size (cm)	38x34x15
Working characteristics	Rated power	50 W
	Working power	15 W
	Working frequency EMS	10-40Hz
	Pulsed microcurrent EMS	50mA
Electrical characteristics	Rated voltage	220V ~ 240V
	Frequency	50Hz/60Hz
	Rated power	50 W

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